V 5.10/02-1

 Gebrauchsanweisung Instructions Mode d´emploi Instrucciones de empleo Istruzioni Instruções de emprego Bruksanvisning Brugsanvisning Οδηγίες χρήσης Instrukcja Pokyny Navodila Pokyny Utmutatú Инструкция по использованию	 SERION ELISA <i>control</i>
 	
0197*	virion\serion
* für BR137A , BR137C , BR1371A , BR1371G , BR1372A , BR1372G , BR109G , BR109M , BR109AVID , BR129G , BR129M , BR129AVID , BR110G , BR110M , BR110AVID	
Hersteller / Manufacturer / Fabricant / Fabricante / Fabbriicante / Fabrikant / Tillverkare / Vyrobc / Πρωτομνηνογούρς / Producent / Výrobca / Proizvajačec / Výrobce / Gyártó / Производитель	
Institut Virion\Serion GmbH Friedrich-Bergius-Ring 19 D - 97076 Würzburg, Germany Tel. +49 (0)931 30 45 0 Fax +49 (0)931 30 45 100 E-Mail: dialog@virion-serion.de Internet: www.virion-serion.de	

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Human serum (lyophilised); Preservative: < 0.1 % Sodium azide.

Intended Use

SERION ELISA *control* are positive control sera for quantitative antibody determination when using immunoassays of the SERION ELISA *classic* product line. The lyophilised human sera are, in addition to the controls supplied with the test kits, supplementary reagents to determine validity of SERION ELISA *classic* test runs, precision and constancy of the method. SERION ELISA *control* are particularly recommended as an aid for internal quality management in accredited laboratories.

Stability and Storage

SERION ELISA *control* are lyophilised. Stability when stored at 2-8 °C: 10 years as of production, expiry date see label; Stability after reconstitution at 2-8 °C: 3 months; Stability after dilution with SERION ELISA *classic* dilution buffer at 2-8 °C: 1 year.

Precautions

Though all SERION ELISA *control* were found negative for HBs-Ag-, HCV- and HIV-anti-

 	
0197*	virion\serion
* für BR137A , BR137C , BR1371A , BR1371G , BR1372A , BR1372G , BR109G , BR109M , BR109AVID , BR129G , BR129M , BR129AVID , BR110G , BR110M , BR110AVID	
Hersteller / Manufacturer / Fabricant / Fabricante / Fabbriicante / Fabrikant / Tillverkare / Vyrobc / Πρωτομνηνογούρς / Producent / Výrobca / Proizvajačec / Výrobce / Gyártó / Производитель	
Institut Virion\Serion GmbH Friedrich-Bergius-Ring 19 D - 97076 Würzburg, Germany Tel. +49 (0)931 30 45 0 Fax +49 (0)931 30 45 100 E-Mail: dialog@virion-serion.de Internet: www.virion-serion.de	

bodies, they should be considered potentially infectious. The product contains sodium azide for preservation. Swallowing and contact with skin or mucous membranes must be avoided!

Target Values and Target Ranges

In comprehensive validation studies, target values and target ranges for each test-specific SERION ELISA *control* are determined in accordance with the new quality control guidelines of the German Medical Association of the year 2008 and are documented on lot-specific certificates of analysis. These are recommended as reference values for the quality management in accredited laboratories when using SERION ELISA *control* with SERION ELISA *classic*. SERION ELISA *control* generally show positive results in test runs with SERION ELISA *classic*. However, a few SERION ELISA *classic* were evaluated in consideration of a so-called clinical threshold, e. g. in order to exclude high seroprevalences. Therefore, the evaluation of individual SERION ELISA *control* might deliver borderline results in exceptional cases.

Preparation

Reconstitution with distilled water as described on the corresponding certificate of analysis.

Performance

Dilution of reconstituted serum with SERION ELISA *classic* dilution buffer as described on the certificate of analysis of SERION ELISA *control*. SERION ELISA *control* are recommended to be used in each SERION ELISA *classic* test run. SERION ELISA *control* are recommended for use with the corresponding SERION ELISA *classic* test. Please refer also to the instruction manual of the SERION ELISA *classic* test. SERION ELISA *control* can be used with other tests only after validation by the user.

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